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(Válida únicamente en los EE.UU.)

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ContactTherapy™ 10

10 Point True Contact™ Back Massager

**Instruction Manual and
Warranty Information**

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2 year
limited warranty

IMPORTANT SAFEGUARDS WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING DANGER- TO REDUCE THE RISK OF ELECTROCUTION:

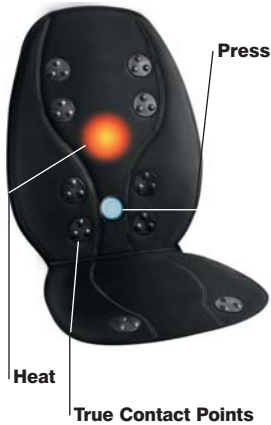
- ALWAYS unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or in shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink. DO NOT place in or drop into water or other liquid.
- NEVER use pins or other metallic fasteners with this appliance.
- Keep dry - DO NOT operate in a wet or moist condition.

WARNING - TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS:

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.

- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- Keep cord away from heated surfaces.
- NEVER use while sleeping or fall asleep while using the massager.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT carry this appliance by supply cord or use cord as handle.
- To disconnect, turn all controls to the "off" position, then remove plug from outlet.
- This appliance is designed for personal, non-professional use only.
- DO NOT use outdoors.
- Temperatures sufficiently high to cause burns may occur regardless of the control setting. DO NOT use on an infant or invalid or on a sleeping or unconscious person. DO NOT use on insensitive skin or on a person with poor blood circulation. Check the skin in contact with the heated area of the appliance frequently to reduce the risk of blistering.
- DO NOT crush. Avoid sharp folds.

SAVE THESE INSTRUCTIONS

Pressure Activated Switch

CAUTION — PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- This unit should not be used by children or invalids without adult supervision.

Please Note:

For your safety, this product is equipped with a pressure activated switch to control the heat function. **Heat will not operate unless pressure is applied to this switch, located on the back cushion of the unit.** Once pressure is removed, heat will turn off. After applying pressure, please allow 5-10 minutes for heat to reach maximum temperature.

Automatic Shut Off

We've designed this back massager for convenience and safety. As a special precaution against falling asleep while using the unit, or leaving the unit on while it's still in operation, we've programmed it to automatically stop massaging 15 minutes after receiving your last command. To reactivate the massage cushion, simply press any button. Any time you change massage functions, the timer will reset. This special feature should not be considered as a substitute for OFF. Always remember to turn your unit off when not in use.

Please Note:

Not recommended for use on wooden surfaces as the zipper may damage the wood. Caution is also recommended when using on upholstered furniture.

Caution:
All servicing of this massager must be performed by authorized HoMedics service personnel only.

Instructions for Use

For Use in Home or Office

Plug adapter into a 120 volt AC outlet and plug the receptacle end into the input jack, located along the side of the cushion. Attach the massage cushion to any chair for a luxurious massage as you read, rest or even work.

For Use in Your Car

Simply plug the 12 volt auto adapter into a cigarette outlet and plug the receptacle end into the input jack, located along the side of the cushion. Slip the massage cushion straps over your seat to enjoy a relaxing massage as you drive.

Strapping System

The Contact Therapy Back Massager incorporates a unique strapping system which allows you to fasten it to most any chair or auto seat. Just slip the straps over the seat or chair and adjust the convenient hooks to fit. Your massager won't slip or slide away.

Maintenance

To Store Place massager in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit. Do not hang the unit by the controller cord.

To Clean Unplug the unit and allow it to cool before cleaning. Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, paint thinner to clean.
- Do not attempt to repair the massager. There are no user serviceable parts. For service, send the unit to the HoMedics address listed in the warranty section.

Power Adapters

This Back Massager is powered by a custom 120 volt AC home adapter, and 12 volt DC auto adapter (cigarette lighter plug-in style). Both are included with your unit.

True Contact Points

The Contact Therapy 10 message cushion targets specific trouble spots throughout the shoulders, back and thighs for focused relief. Massaging nodes are located on each contact point to give a concentrated massage to those areas.

Note: When selecting the heat feature with massaging action, there will be a slight decrease in the strength of the massage output. This adjustment is perfectly normal and should not be interpreted as a defect.

Heat

Soothing, penetrating heat is activated with this button. To turn off heat, simply press the button again. A glowing L.E.D. light indicates that heat is on. PLEASE NOTE: The heat option will only work while a massage feature is selected.

Targeted Massage Zones

Activate contact points to target specific areas: shoulders, back, or thighs. Customize your massage program by choosing a specific area or all three at once. To select a zone press the ZONE SELECT button and continue to press until you have reached the desired zone. ZONE SELECT starts with all three zones and will cycle through in this order: All Zones, Upper Back, Lower Back, Thighs and then starts over again with All Zones. When you select any or all of the three zones, the corresponding L.E.D lights will illuminate.

ContactTherapy 10

10 Point True Contact Back Massager

On/Off

To turn on the massage functions, first press the POWER button. The L.E.D indicator will illuminate and confirm your selection. The massage cushion will be in the Low (L) position when first turned on. To turn off the massage functions, simply press the POWER button until the L.E.D lights turn off (the massage cushion will cycle through medium and high (H) before turning off).

Massage Strength

Choose the intensity of your massage. The Contact Therapy gives you three options of massage strengths: low (L), medium and high (H). L.E.D. lights correspond to your selection. To change the strength of your massage, simply press the POWER button again until you reach your desired intensity.

Program Styles

The Contact Therapy incorporates 3 exclusive preprogrammed functions that can be easily selected by pressing the program button. Move from one function to another by simply pressing the program button again. All 3 programs cycle through the body zones.

Steady Massage:

Contact points massage all over.

Traveling Massage:

Massage runs down your shoulders, through the back to your thighs and back up.

Pulsating Massage:

All 10 contact points turn on and off, giving you an intense pulsating massage.

TWO YEAR LIMITED WARRANTY
(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of two years from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of two years from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics' option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com

©2004 HoMedics, Inc. and its affiliated companies, all rights reserved.
HoMedics® is a registered trademark of HoMedics Inc. and its affiliated companies.
The beauty of living well.™, Contact Therapy™ and True Contact™ are trademarks of HoMedics Inc. and its affiliated companies.

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All rights reserved.