HOMEDICS®

#1 BRAND IN MASSAGE*

PLEASE TAKE A
MOMENT NOW
REGISTER YOUR PRODUCT AT:

www.homedics.com/register

Your valuable input regarding this product will help us create the products you will want in the future.

CORDLESS MASSAGING CAPE

WITH VIBRATION & SOOTHING HEAT

INSTRUCTION MANUAL AND WARRANTY INFORMATION

2-YEAR LIMITED WARRANTY

HCM-WRP590H

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING.

DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or in a shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink.
- DO NOT place in or drop into water or other liquid.
- NEVER use pins or other metallic fasteners with this appliance.

WARNING – TO REDUCE THE RISK OF BURNS, FIRE, ELECTRIC SHOCK, OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically, any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a HoMedics Service Center for examination and repair.
- Keep the cord away from heated surfaces.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- DO NOT carry this appliance by the supply cord or use cord as a handle.
- To disconnect, turn all controls to the off position, then remove plug from outlet.
- DO NOT use outdoors.
- Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

- NEVER operate on a soft surface such as a bed or couch where the air openings may be blocked.
- A charger that is suitable for one type of battery pack may create a risk of fire when used with another battery pack. The unit should only be charged by adapters with 5VDC-2A or less output.
- DO NOT expose a battery pack or appliance to fire or excessive temperature. Exposure to fire or temperature above 265°F (130°C) may cause explosion.
- Follow all charging instructions and do not charge the battery pack or appliance outside of the temperature range specified in the instructions. Charging improperly or at temperatures outside of the specified range may damage the battery and increase the risk of fire. Product operating and charging range: 32°F – 104°F (0°C – 40°C).
- Use only with Listed/Certified Information Technology (Computer) Equipment, ITE Power Supply or Class 2 Power Unit.

CAUTION: All servicing of this massager must be performed by authorized HoMedics service personnel only.

SAVE THESE INSTRUCTIONS

CAUTION – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- Consult your doctor prior to using this product, if:
 - You are pregnant
 - You have a pacemaker
 - You have any concerns regarding your health
- Not recommended for use by diabetics.
- NEVER leave the appliance unattended, especially if children are present.
- DO NOT use this product for more than 30 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- This product is equipped with an overheating protection circuit. If the power is cut off, please wait until the product is cooled down.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies.
- This unit should not be used by children or invalids without adult supervision.
- This product is not for use in automobiles.
- This product is intended for household use only.

INSTRUCTIONS FOR USE

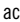
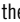
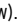
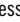
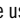
Fig. 1

Fig. 2

Fig. 3

1. Insert USB cord into the port on the rechargeable battery pack. The USB cord is located in the fabric pouch on the inside lining of the cape (Fig. 1).
2. Touch the power  button on the battery pack once to turn the battery pack on. The blue indicator lights will illuminate to indicate battery level.
3. Place unit around your shoulders (Fig. 2).
4. To activate the massage action, press the vibration  button on the control pad. The vibration massage will activate on the lowest setting. To increase the massage intensity, press the vibration  button again. To turn the vibration massage off, press the vibration  button a third time.
5. To activate heat function, press the heat  button on the control pad once for high heat (red). Press again for low heat (yellow). To turn off the heat, press the heat  button a third time (Fig. 3).

NOTE: Heat and vibration can be used independently or combined.

The battery pack should arrive with a full charge. If the battery is depleted, you may plug the provided adapter cord into the port on the battery pack and plug the other end into a USB adapter or outlet. The blue indicator lights will illuminate in sequence to indicate active charging. The product will be fully charged after approximately 3 hours of charge time. With a full charge, the battery pack will provide power for up to 2 hours.

MAINTENANCE

TO CLEAN

Unplug the unit and allow it to cool before cleaning.

- DO NOT attempt to repair the massager. There are no user-serviceable parts. For service, contact Consumer Relations at the number listed in the Warranty section.
- This product has an internal, non-replaceable lithium-ion battery. This battery is not user-serviceable. Please dispose in accordance with local, state, province, and country regulations.
- Machine washable.

WASHING INSTRUCTIONS

- Remove battery pack before washing.
- DO NOT dry clean or bleach.
- Presoak for 15 minutes in mild soap and cold water.
- Machine wash in mild soap and cold water on slow cycle for 2 minutes.
- Rinse in fresh, cold water.
- Spin dry. Stretch to original size.
- DO NOT put through a wringer.

DRYING INSTRUCTIONS

- For best results, air dry over 2 parallel lines.
- DO NOT use clothes pins.
- Machine dry caution: Do not use commercial dryers or dryers at your local laundromat.
- Preheat dryer on low temperature setting for 2 minutes.
- Add cape and tumble dry for 10 minutes.
- Remove while still damp and stretch to original size.
- Air dry over a line or shower rail.

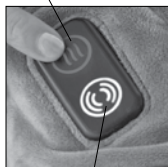
TO STORE

Place cape in its box or in a cool, dry place. Avoid contact with sharp edges or pointed objects that might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit. Do not hang the unit by the power cord. The product should be stored at 77°F (25°C).

NOTE: This unit is equipped with a 30-minute auto-off feature for your safety.

Heat

Vibration

Battery Pack

FCC STATEMENT

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: HoMedics is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

#1 BRAND IN MASSAGE*

2-YEAR LIMITED WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 2 years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to retailers.

To obtain warranty service on your HoMedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

HoMedics does not authorize anyone, including but not limited to retailers, the subsequent consumer purchaser of the product from a retailer, or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse, accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved, and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL, OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS THAT ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged, and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof that are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights that may vary from state to state and country to country. Because of individual state and country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit www.homedics.com. For Canada, please visit www.homedics.ca.

FOR SERVICE IN THE USA

Email: cservice@homedics.com

8:30am–7:00pm EST Monday–Friday

1-800-466-3342

FOR SERVICE IN CANADA

Email: cservice@homedicsgroup.ca

8:30am–5:00pm EST Monday–Friday

1-888-225-7378

*Source: The NPD Group. January 2018–December 2018, Retail Tracking Service.

©2019 HoMedics, LLC. All rights reserved.

HoMedics and HoMedics #1 Brand In Massage are registered trademarks of HoMedics, LLC.

Distributed by HoMedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390

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©2019 HoMedics, LLC. Todos los derechos reservados.
HoMedics y HoMedics #1 Brand In Massage son marcas registradas de HoMedics, LLC.
Distribuido por HoMedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390